# VueTip® Trocar Swab Accessory Kit

## Laparoscopic Solutions

**Indications For Use:** VueTip® Trocar Swab Accessory Kit is a single-use, sterile device to be used to clean a trocar cannula or wipe a lens during laparoscopic or endoscopic procedures.


This document is designed to assist in using this product. It is not a reference to surgical technique.

This equipment is intended for use by trained healthcare professionals only.

**For single-use only. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing, or re-sterilization may lead to malfunction or risk of cross-contamination which in turn may result in patient injury, illness or death.**

Report serious incidents involving this device to Buffalo Filter at [bf.customerfeedback@filtrationgroup.com](mailto:bf.customerfeedback@filtrationgroup.com) or to your local distributor. In addition, European customers should also report to [europa@emergogroup.com](mailto:europa@emergogroup.com) and the competent authority in your member state.

### Contents

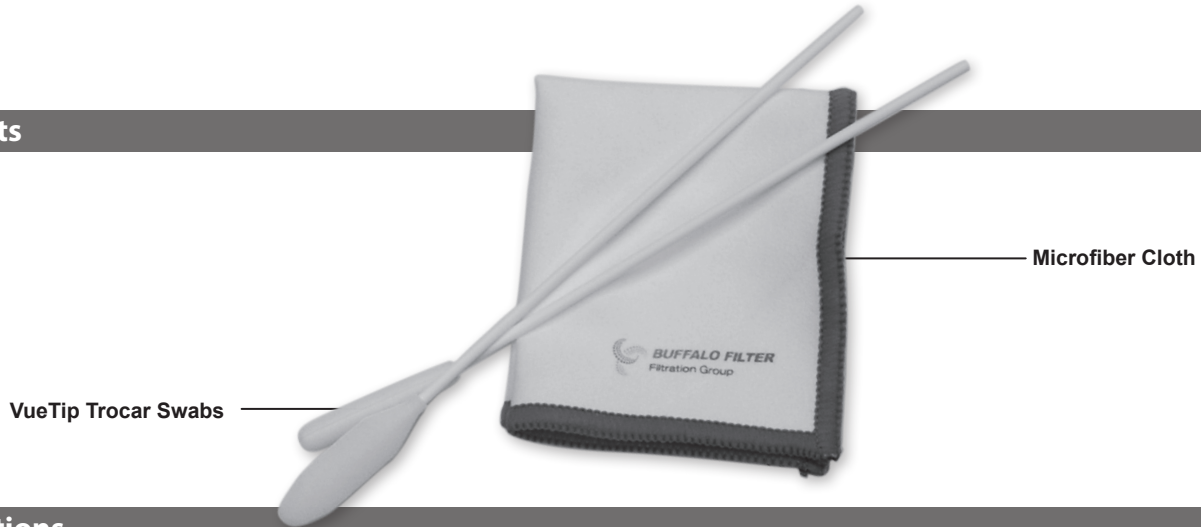### Instructions

#### VueTip Trocar Swab Accessory Kit

Contains a Microfiber Cloth and two VueTip Trocar Swabs for 3-7mm and 7-12mm trocar cannulas.

#### Microfiber Cloth

The non-abrasive Microfiber Cloth can be used to clean the scope lens.

#### VueTip Trocar Swabs are used to Clean Trocar Cannula

1. Use the small head VueTip Trocar Swab for 3–7mm trocars, and the large head VueTip Trocar Swab for 7–12mm trocars.
2. Grasp handle and push foam head straight into the trocar. Do not release or bend the VueTip Trocar Swab.
3. On certain trocar models that have a spring loaded valve, open the valve during insertion or retraction of the VueTip Trocar Swab. Do not extend foam head beyond distal end of trocar.
4. VueTip Trocar Swabs and Microfiber Cloth are radiopaque.
5. VueTip Trocar Swabs are not compatible with Snowden-Pencer trocars or other metal trocars.

**Manufactured by:**

Buffalo Filter LLC  
5900 Genesee Street, Lancaster, NY 14086  
U.S. toll-free 800.343.2324 · International: 1.716.835.7000 · Fax: 716.835.3414  
Email: [smoke-info@buffalofilter.com](mailto:smoke-info@buffalofilter.com) · [www.buffalofilter.com](http://www.buffalofilter.com)

904486 Rev B 02/02/2018

|   |   |   |   |   |   |   |
|---|---|---|---|---|---|---|
|          |  |  |  |  |  |  |
|          |  |  | Not Made with Natural Rubber Latex  |   |  |  |
| Consult Symbol Glossary at <a href="http://www.buffalofilter.com">www.buffalofilter.com</a> |   |   |   | Emergo Europe<br>Prinsessegracht 20<br>2514 AP The Hague<br>The Netherlands           |   |  |
|   |   |   |   |   |   | VueTip01<br>VueTip10  |

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**Manufactured by:**

Buffalo Filter LLC  
5900 Genesee Street, Lancaster, NY 14086  
U.S. toll-free 800.343.2324 • International: 1.716.835.7000 • Fax: 716.835.3414  
Email: [smoke-info@buffalofilter.com](mailto:smoke-info@buffalofilter.com) • [www.buffalofilter.com](http://www.buffalofilter.com)

904486 Rev B 02/02/2018

|   |   |   |   |   |   |               |   |
|---|---|---|---|---|---|---------------|---|
|          |  |  |  |  |  | 0120          | STERILE R   |
|          |  |  | Not Made with Natural Rubber Latex  |   |  | 40°C<br>104°F |  |
| Consult Symbol Glossary at <a href="http://www.buffalofilter.com">www.buffalofilter.com</a> |   |   |   |   | Emergo Europe<br>Prinsessegracht 20<br>2514 AP The Hague<br>The Netherlands           | 4°C<br>40°F   |   |

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